**Disposable EEG [Electroencephalography]  
Cup Electrode**
**Intended Applications**

EEG, EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

**Sterilization**

Non-sterile.

**Caution**

Disposable EEG Cup Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

**Packaging**

Ten Electrodes per pouch.

**Instructions for Use**

Clean application site. Apply Cup Electrode using conductive gel, paste, collodion or other similar adhesive. When finished, remove Electrode and clean application site. This product is single-patient use only. Discard Electrode after use.

Symbol	Indication
	Lot Number
	Do Not Reuse
	Date of Manufacture
	Temperature Limitations
	Keep Away from Sunlight
	No Latex Used
	Disposable Cup Electrodes are in conformity with the appropriate European Medical Directive

**Manufacturer**  
RhythmLink International, LLC  
1140 First Street South  
Columbia, SC, USA 29209-3540  
+1.866.633.3754 [toll-free]  
+1.803.252.1222  
+1.803.252.1111 [fax]  
sales@rhythmink.com  
Rhythmink.com

**European Authorized Representative**  
Mdi Europa GmbH  
Langenhagener Str.71  
30855 Hannover-Langenhagen

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