English

### Disposable Sticky Pad™ Surface Electrode

#### Intended Applications

EEG [Electroencephalography], EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], NCS [Nerve Conduction Studies], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

#### Sterilization

Non-sterile.

#### Caution

Sticky Pad Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

#### Packaging

Stimulating and Recording Pads: four Electrodes per pouch. Ground Pads: one Electrode per pouch.

#### Instructions for Use

Select appropriate Sticky Pad Electrode. Prep application site by removing excess hair, oils and dirt. Remove Mylar backing from Electrode and place on application site. Apply pressure to center of Electrode and move to edges. Remove by pulling directly on Electrode. DO NOT PULL ON THE CABLE! Remove remaining hydrogel with clean, soapy water. This product is single-patient use only. Discard Electrode after use.

Symbol	Indication
	Lot Number
	Use Before
	Do Not Reuse
	Date of Manufacture
	Temperature Limitations
	Keep Away from Sunlight
	No Latex Used
	Sticky Pad Electrodes are in conformity with the appropriate European Medical Directive

**Manufacturer**  
RhythmLink International, LLC  
1140 First Street South  
Columbia, SC, USA 29209-3540  
+1.866.633.3754 [toll-free]  
+1.803.252.1222  
+1.803.252.1111 [fax]  
sales@rhythmink.com  
Rhythmink.com

**European Authorized Representative**  
Mdi Europa GmbH  
Langenhagener Str.71  
30855 Hannover-Langenhagen

**Caution:** Federal Law [USA] restricts this device for sale by or on the order of a licensed medical practitioner. Rhythmink® is a registered trademark of Rhythmink International, LLC. Sticky Pad™ is a trademark of Rhythmink International, LLC.

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