English

## Disposable Subdermal Needle Electrode

### Intended Applications

EEG [Electroencephalography], EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], NCS [Nerve Conduction Studies], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

### Sterilization

Sterilized by Ethylene oxide.

### Caution

Subdermal Needles are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product. Product is sterile only if packaging is unopened. When selecting a Subdermal Needle Electrode, it is important to choose the appropriate length and diameter.

### Packaging

Single Needles: one Electrode per pouch. Paired Needles: two electrodes per pouch.

### Instructions for Use

Select appropriate Subdermal Needle Electrode. Prior to use, remove Needle sheath. When introducing Electrode into patient, do not insert up to hub. If needle bends before, during or after insertion, do not straighten or re-insert. Discard and replace with a new Electrode. This product is single-patient use only and should not be sterilized or reused. Discard Electrode after use.

RhythmLink International, LLC  
1140 First Street South  
Columbia, SC, USA 29209-3540  
+1.866.633.3754 [toll-free]  
+1.803.252.1222  
+1.803.252.1111 [fax]  
sales@rhythmlink.com  
RhythmLink.com

**European Authorized Representative**  
Mdi Europa GmbH  
Langenhagener Str.71  
30855 Hannover-Langenhagen

**Caution:** Federal Law [USA] restricts this device for sale by or on the order of a licensed medical practitioner. RhythmLink® is a registered trademark of RhythmLink International, LLC.

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